

Trainhaler™

Trainhaler is the first simple pMDI training system to provide flow and co-ordination coaching.

“Successful training in inhaler technique depends upon effective communication or proper technique and its purpose, and monitoring to ensure that the skills have been learned and retained,” (1)

Product Information

Trainhaler is a placebo-like pMDI training device. With the addition of a Flo-Tone Trainer, it becomes the essential tool for healthcare professionals to teach inhaler technique (flow and co-ordination).

The Trainhaler is suitable for multi-patient use, with the Flo-Tone Trainer mouthpiece being replaced for individual patients.

When the patient inhales from the Trainhaler Training system, they will hear a whistle that confirms their flow rate is adequate. The start of the whistle sound is the signal to press the canister (they will hear a noise similar to a pMDI). The whistling sound should be maintained for 5 seconds, if possible. This ensures that the patient is inhaling gently, at the correct flow rate.

What is this Product used for?

Trainhaler is designed to be used by healthcare professionals to coach patients, helping them to improve their inhaler technique.



Product Features and Benefits

- Contains no medication (placebo-like)
- No HFA (but sounds the same)
- For multi-patient use (can be cleaned easily)
- Flow rate guidance with Flo-Tone attachment (whistle)
- Firing signal for easy co-ordination when Flo-Tone device is attached
- Low-cost option for inhaler technique training
- Teaches correct flow rate, with the ability to master inhale/actuate co-ordination
- Simple, non-electronic training device, can be used anywhere

Product Suitability

Hospital



Clinic



Technical Specifications

Dimensions: 60 x 35 x 95mm

Weight: 30g

Part Code

Trainhaler with Flo-Tone Trainer (pack of 1 each): 3152060

In-Check Flo-Tone Trainer Regular (pack of 10): 3152002

Flo-Tone Accessories

In-Check Flo-Tone Trainer

The device is to be attached to Trainhaler or a placebo pMDI training inhaler. The device is supplied in boxes of 10.




Instructions of how to use the device are printed on the box in four different languages:

- English
- French
- German
- Spanish

These instructions can also be obtained from the following website: www.flo-tone.com

In-Check Flo-Tone

In-Check Flo-Tone Trainer: fits Trainhaler and regular (barrel-shaped) inhaler mouthpieces.

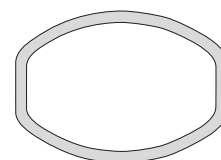

Ref 1: Sanchis J, Corrigan C, Levy ML, Inhaler Devices: From Theory to Practice. Respiratory Medicine (2013).