

## WARNING!

With any inhalation device it is important to check for loose foreign objects before the device is used. The transparent material used in the construction of the In-Check enables the user to make a visual check before inhalation.

**To prevent choking, please make sure there are no foreign objects in the device before use.**

Damage has been caused to In-Check meters through the use of excessive force when trying to shake water from the meter after cleaning. Please ensure that the meter is shaken by holding the ROUND END only – to hold the other end and shake the meter can result in the resetting weight damaging the piston & spring assembly.

HOW TO SHAKE WATER OUT OF IN-CHECK



HOW NOT TO SHAKE IN-CHECK  
**DAMAGE TO THE IN-CHECK  
CAN RESULT !!!**



Occasionally, the resetting weight is not easily dislodged. In such cases, bang the rounded end of the meter against a hard surface (such as a table-top, although not if it is made of glass). The In-Check body was designed to withstand such attempts to dislodge the resetting weight.